




Flex Hoof Boots

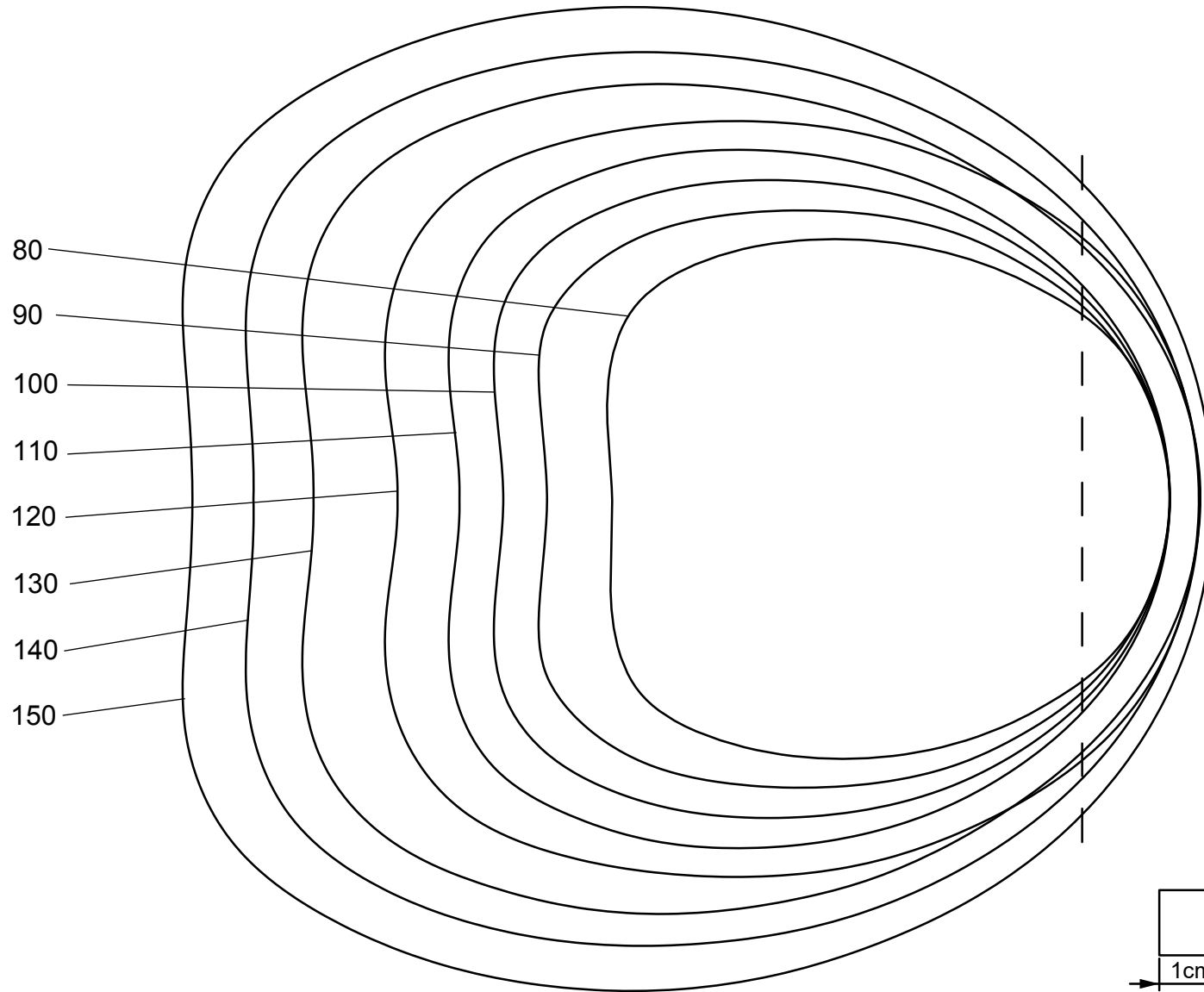

Flex Pad Instructions

10mm thick pads



Flex Pad Terra

Cut the pad to size following the outline.

Ensure the hoof's toe seats fully into the boot's toe area; a gentle tap with a rubber mallet can help.

The pad may expand slightly beyond the hoof edge. Side expansion is normal and can improve fit, but trim any excess from the toe area to maintain a correct breakover.

Care:

Brush off dirt or wash with warm water.

Flex Pad EVA Relief

Cut the pad along the straight line at the toe area. This helps the hoof seat correctly in the boot and maintains a biomechanically correct breakover.

Flex Boots feature one of the most anatomically correct breakover points on the market, supporting the horse's natural movement.

Care:

Clean with a brush and/or water. EVA pads may be washed with lukewarm water; plumpness can be restored by drying them in a warm place or briefly in a tumble dryer. Do not leave them in the dryer for too long.